

7D JULATI (TIME SHUTTLE)

Product Manual

* WHEN OPERATING THIS PRODUCT, PLEASE READ THE MANUAL OF THIS INSTRUMENT CAREFULLY



Preface

First of all, I would like to express my heartfelt thanks to the customers who purchased our instruments. This manual is prepared for customers to better use this instrument, in order to use the instrument correctly, to prevent damage to the operation and users, protect the instrument from damage, to avoid causing unnecessary property losses, please read the manual carefully before use. The Company considers itself responsible for the safety and reliability of the instrument only if the fitting, reassembly, improvement and maintenance are performed by personnel approved by the Company, the electrical equipment connected to the instrument conforms to national standards, and the instrument is used in accordance with the operating instructions. This is a kind of life beauty instrument. If the beauty organization using this instrument can not operate the instrument normally, it may cause the instrument function failure, and may endanger the health of the person. The Company will provide, for a fee upon request, a circuit diagram, along with a list of calibration methods and other information to assist the Customer in the maintenance of the parts of the instrument that the Company classifies as servable by the Customer. The product must be used in strict accordance with the usage method in this manual. The company shall not assume any responsibility for any consequences caused by misuse of methods other than the manual. The Company makes no warranties of any kind, including (but not limited to) any warranties of merchantability and fitness for a particular purpose. The Company shall not be liable for errors contained in this material or for incidental or consequential damages arising from the provision, actual performance or use of this manual. This user manual contains all the basic information of the instrument. This is the starting point for new users to start using this instrument, because it introduces the basic system, explains how it works, introduces the instrument installation and various operating methods, you can find the required information through the directory. This manual contains proprietary information protected by patent law. All rights reserved. Photographs, photocopies, reproductions or translations of any part of this manual into other languages are not allowed without the written consent of the Company. The contents contained in the manual are subject to change without notice to the user.

product list

1. Introduction to the instrument	
1.1 Principle of instrument operation.....	03
1.2 Advantages of the instrument... ..	03
1.4 Appearance of the instrument... ..	04
1.5 Interface description.....	05
2. Precautions for power on	
2.1 Handling safety..... ..	06
2.2 Electrical safety... ..	06
3. Operational safety..... ..	06
4. Operating instructions..... ..	07
5. Contraindications... ..	07
6. Common faults and troubleshooting methods.....	07
7. Warranty card.....	08

1. INSTRUMENT INTRODUCTION

1.1 The working principle of 7D Julati (Time Shuttle)



The 7D Zulati is the third generation of Korea's Kolesis. Utilizing the patented technology of HFI and MNFU, it generates high-energy focused ultrasound, which quickly penetrates the epidermis, the fat layer, and the membrane of the SWS layer shrinks, instantly causing the skin tissue to reach a high temperature of 6~T5C, so that the skin produces a thermal coagulation effect. , T- he fascia layer forms a net-like condensation point, which stimulates the shrinkage and firmness of the fiber and collagen tissue, regeneration and repair, and achieves layer by layer lifting and firmness. The microscopic high-energy focused ultrasound system of 7D Poly Lati has a smaller focus point than other HIFU equipment, and more accurately transmits high-energy focused ultrasound energy at 65~75°C to the target skin tissue layer, producing thermal coagulation effect without harming the surroundings organization. While stimulating the proliferation of collagen and elastic fibers, it greatly improves comfort

1.2 Instrument advantages


1. Non-surgical treatment of facial lifting, firming and body contour shaping system;
2. Won the US FDA and EU CE international safety certification;
3. The world's latest MMFU dual-core high-energy focused ultrasound technology;
4. Non-invasive, safe, fast, comfortable and effective;
5. Layered anti-aging seventy 1.5mm, 2.0mm, 3.5mm, 4.5mm;
- 6, 6mm, 9mm, 13mm probe depth, can fully deal with skin aging problems;
7. The 2.0mm probe is unique in the world and can be operated without any restricted area;
8. The triple guarantee of one lifting, two firming, and three metabolism fat-dissolving, one machine can get it;
9. High profit, high consumption, high satisfaction and high repurchase.

1.3 Instrument appearance and interface description



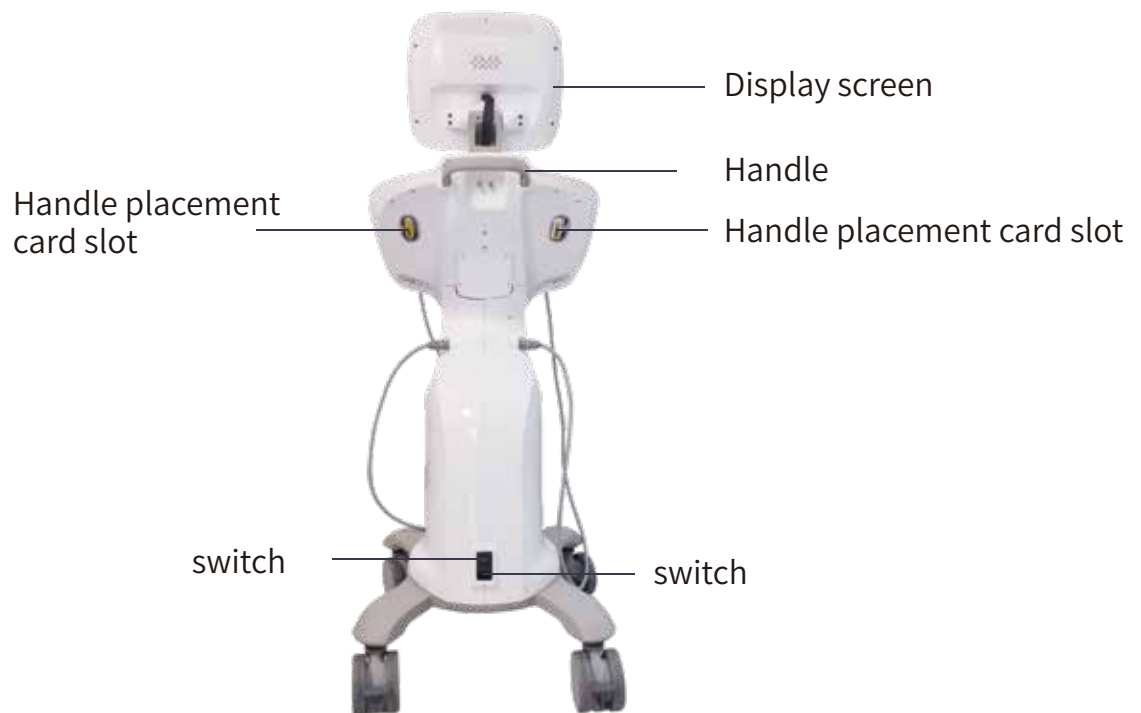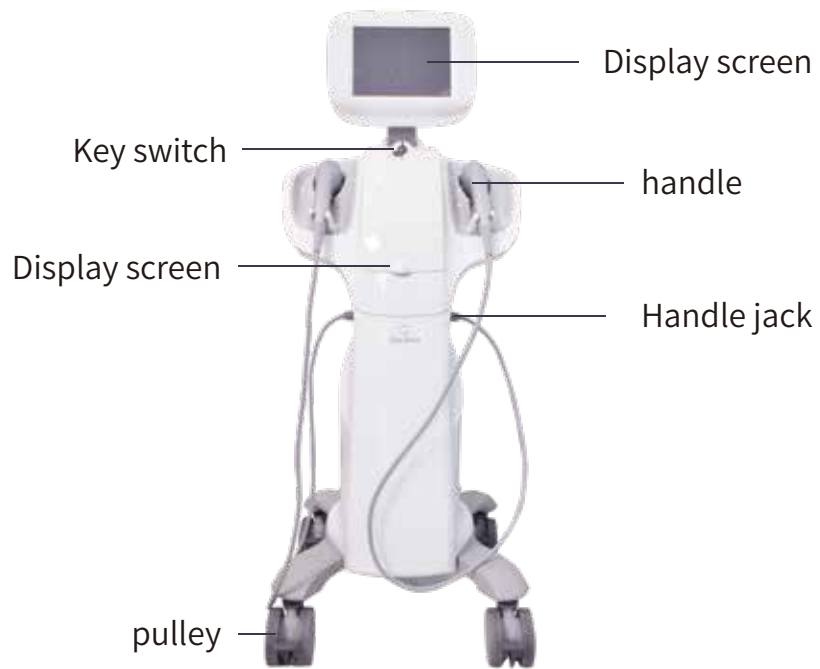

1.4 Interface description

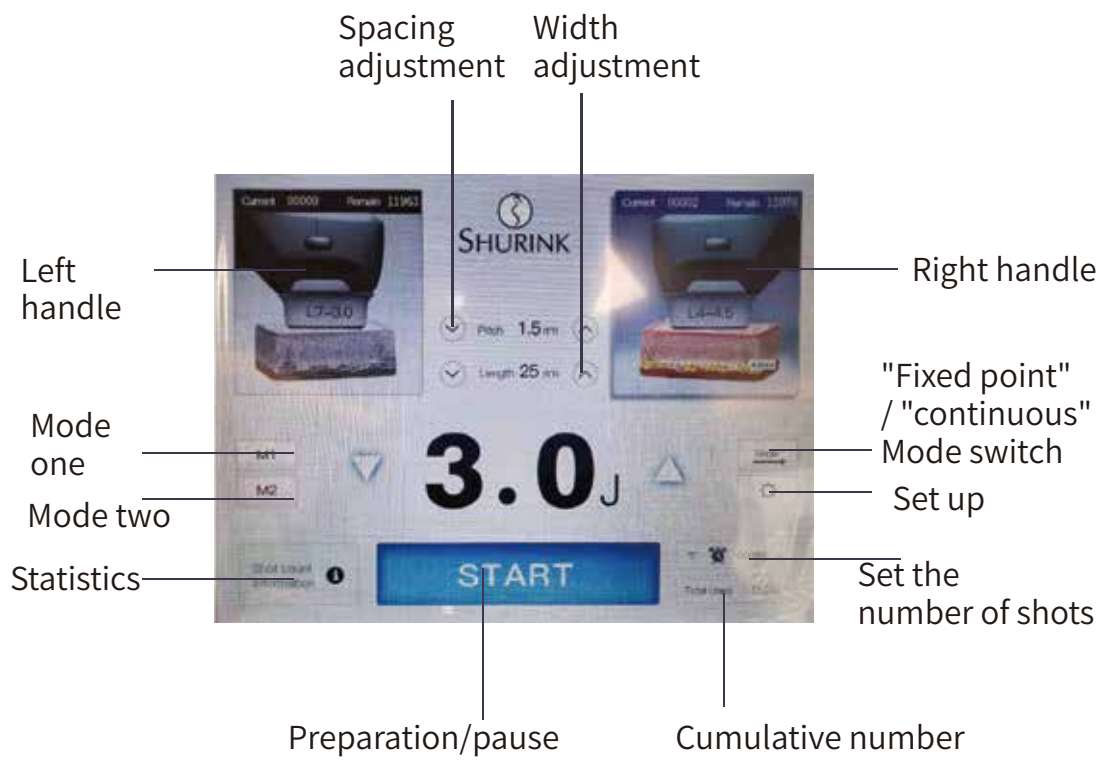

2. PRECAUTIONS FOR POWER ON

2.1 Safe handling

This operating system has precision instruments in memory, so users should try not to move this machine long distances. The shock-absorbing packaging provided by the original manufacturer must be used in the process, otherwise it may cause the optical system to be out of adjustment and affect the energy output.

2.2 Electrical safety

This operating system uses a single-phase 220V power supply, and the maximum output current is not more than 10A. In order to extend the service life of the machine and reduce the impact of power grid fluctuations on the machine, it is recommended that users refer to the use of a 3000w voltage stabilizer.

3. OPERATIONAL SAFETY

- 1) This instrument is dedicated for beauty salons and is not suitable for home users. The instrument needs to be operated by a professional beautician. Non-professional operation may cause unnecessary injury.
- 2) When you receive the instrument, please check all the accessories and check whether the machine is damaged. Refer to the list of instrument accessories. If the machine is damaged during transportation, please inform the factory.
- 3) The power supply used should be consistent with the specified power supply value marked on the machine, otherwise the machine may not work or even burn out the machine. This machine is grounded through the ground wire in the three-core power supply. Good grounding is essential for safe operation.
- 4) Please keep the original packaging of the instrument. When you need our assistance, you must wrap the machine in the original packaging before sending it for repair.
- 5) Do not disassemble the company's instrument or attempt to perform operations not described in this manual. All maintenance work must be performed by professional maintenance personnel approved by the company.
- 6) Do not install or operate the instrument near a water source or when your hands are wet, and be careful not to spill liquid on the instrument.
- 7) Do not place anything on the wire, and do not install the instrument in a place where the power cord is easy to be stepped on.
- 8) If the following situations occur, please unplug the instrument from the power socket and contact a professional service station.
 - a) Liquid splashed on the instrument, or the instrument was exposed to rain or water:
 - b) The instrument emits peculiar smell, smoke or abnormal noise

4. OPERATION INSTRUCTIONS

1. Preparation stage:

- 1) Selection of nursing items: According to the conditions and needs of the client's private parts, select the appropriate nursing items.
- 2) Customer file registration: fill in the customer file information in detail, take pictures and archive before treatment, establish customer personal data files, and confirm treatment items.
- 3) Treatment site planning: A treatment plan with prominent key points should be formulated according to the specific situation of the client's privacy problems.
- 4) Introduce the treatment plan: introduce the care plan to the customer in detail, so that the customer fully understands all the occurrences before and after the care Matters needing attention after reflection and nursing.
- 5) Tool preparation: Before operation, prepare all the items needed for operation: small bowl, scraper, absorbent cotton, gel, alcohol cotton, and instrument care accessories.
- 6) Clean the treatment area: clean the skin thoroughly and keep the skin dry: disinfect the hands of the beautician and the bald head of the instrument to ensure the sterility in the operation. (Note: Before treatment, all metal products such as rings, watches and necklaces worn by the customer should be removed)

5. CONTRAINDICATIONS

The following people are not suitable for using this instrument for nursing:

1. Pregnant women, hypertension, heart disease, diabetes
2. It is forbidden for people who are allergic to light
3. People with open wounds
4. Do not do during menstrual period

6. COMMON FAULTS AND TROUBLESHOOTING METHODS

Failure phenomenon	Causes	Exclusions and solutions
Device not working	The power plug is not inserted into the socket The power switch is off Power fuse failure	Check power Check the power switch Replace the fuse
The handle does not work	Is the number of probes used up? The working head is damaged The interface on the back of the instrument is not installed correctly	Check the number of probes Replace working head re-install

7. WARRANTY CARD

1. The machine can be replaced with a new one within 7 days from the date of purchase (with the receipt) if there is a quality problem and the appearance is intact. One machine, the host is guaranteed for one year (except for wearing parts), other non-warranty scope or the warranty period has expired, also Responsible for repairs and updates, and appropriate repair fees - are required. If the instrument fails, it needs to be returned for repair. Within half a year from the date, the buyer shall bear the return freight, and the seller shall bear the freight; the freight for more than half a year shall be borne by the buyer.

2. Free warranty service will not be provided for the failures caused by the following users' personal reasons;

- ① Failure caused by unauthorized disassembly or modification of this product;
- ② Failure caused by careless use;
- ③ Failure caused by lack of reasonable maintenance;
- ④ Failures caused by failure to follow the correct instructions in the instruction manual;

Warranty Card

Name _____ phone _____ Postcode _____

Shipping address _____

product name _____ Purchase date _____

Maintenance records			
Return date	Fault content	Treatment plan	Maintenance company

Remarks: This card should be filled in by the buyer and seller. When returning the instrument for repair, this card must be filled out and sent with the instrument. After the instrument is repaired, the card will be sent back to the buyer along with the instrument. Please keep this card properly at that time.